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Instruction manual wheel chair Caribou

This wheelchair is designed for occasional or frequent use, and can be used indoors and outdoors. The wheelchair is designed for a single user of up to 120kg. The frame accommodate with 24" self propel wheels.

The wheelchair can be manually moved in forward and reverse and is propelled either by the wheelchair user or a carer, depending upon the model of wheelchair in use.

This model has detachable armrest, swing away footrest.

GB

Instructie

This wheelchair frame is easily foldable to facilitate a light carry weight and minimize storage space.

I. Unfolding

Press the side tube of the seat vertically, and then you can unfold the wheelchair. Please see the picture as follows:

II. Folding the wheelchair

Remove the seat and back cushions. Then grab the centre of the front and rear edges of the seat sling. Pull upwards on the seat sling and the wheelchair should naturally fold up.

III. Height Adjustment for the Foot Rest

First loosen the stirrup screw, then slide the foot rest up or down as desired. The fixing screw should be tightened after the adjustment.

IV. Assembly and Disassembly of the Arm Rests

Hold the Arm Rest clip and turn it upward, in order to turn the Arm Rest backwards. This is convenient for the user to enter or leave the Wheel Chair easily. Attention: Don't turn the Arm Rest forcibly when in position. By moving the Arm Rest downward, and pressing it lightly it will lock into position automatically.

GB

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Before Using the Wheelchair read the safety notices below:

- Use slow speeds on gradients. Do not exceed the maximum gradient stated.
 - The wheelchair is only suitable for single occupancy.
 - Keep your feet on the footplates when moving. Do not stand on the footplates.
 - Do not use escalators.
 - Do not reverse down a gradient
- Care and maintenance**
- Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.
 - Do not reach for items further than your arm will extend.
 - Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc.
 - We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring.
 - Ensure that fingers and objects do not get caught in the spokes of the rear wheels.

The transit wheelchair can only be propelled by an attendant. The attendant should push the wheelchair from behind using the handgrips, and following the safety advice above.

The self propel wheelchair can be propelled by an attendant (as directed above) or the wheelchair user. The user can propel themselves by pushing on the handrims attached to the side of the rear wheels, and slow themselves down by applying pressure to the handrims. When stationary the brakes should always be applied.

Brake operation

Parking Brakes

To apply the parking brakes, push the handle on the brake unit until locked in place. To release the brake, push the handle in the opposite direction.

The brakes should always be applied when the wheelchair is stationary.

Care and maintenance

Before each use of the wheelchair, the brakes and tyres should be checked. The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth. The wheelchair comes with a toolkit to perform simple maintenance tasks and most common adjustments.

The user should routinely check the following items.

Tyre wear
Wheel bearings
Castors
Brakes
Legrest locking mechanism
Seat upholstery
Back upholstery
Arm pads
Rear wheel quick release

Specificaties

GW: 24kg
NW: 21.5kg
Weight: 120kg
Dimension: 820*250*930mm
We reserve the right to change designs and specifications.

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PROTENO GmbH
Kaphingst Online AG